

## ThinPrep Analyzer v2.5 Change Log

### New Features:

- Added Current Selection Bar (Orange)
- Added Database Report
- Show "Results" dialog after loading Zips

### \_BUG\_ Fixes:

- Fixed SCARA Tray Placement Retry Chart to show Tray 1 correctly
- New zip extraction method that allows for zips to have different folder structures
- Service History XML having a different time zone will still ingest

### Minor Improvements:

- Select Lab Window - No lab preselected
- Select System Window - No system preselected
- Typing in a serial number will take you directly to that number, NOT two before
- "Removing duplicate records" changed to "Removing any duplicate records"
- Create a log file to log loading and deleting zips (incl. what file(s) didn't load on errors)
- Create an error log file to log all possible ingestion errors

### Other:

- No longer manually change lab names
- All non-alphanumeric chars in lab names are turned into underscores
- XML files with & in the lab name will replace the & with an underscore
- Added instrument serial number to "File Read Error" message
- Error messages show actual exception message

### Known Issues:

- Rarely the ingestion will come across an XML error and not handle it properly and it will notify the user and not ingest the zip (it does NOT crash the tool)